

INSTRUCTIONS FOR USE

1. Product Description

PrestoMed Calcium Hydroxide Paste is a ready-to-use calcium hydroxide paste designed for intracanal medication and pulp capping procedures. It provides antimicrobial activity and promotes healing through its high alkalinity.

2. Composition

- Calcium Hydroxide
- Radiopaque fillers
- Viscous vehicle

3. Intended Use

For professional dental use as an intracanal medicament and pulp capping material.

4. Indications for Use

- Temporary root canal dressing between appointments
- Direct and indirect pulp capping
- Apexification procedures
- Treatment of infected root canals
- Management of internal/external resorption

5. Directions for Use

Application as Intracanal Medicament

1. Dry the prepared root canal thoroughly.
2. Apply PrestoMed Calcium Hydroxide Paste into the canal using a suitable applicator or lentulo spiral.
3. Ensure complete but not excessive filling of the canal.
4. Seal the access cavity with a temporary restoration.

Pulp Capping

1. Apply a small amount directly onto the exposed pulp or deep cavity.
2. Cover with a suitable base or restorative material.

Important:

- Avoid overfilling beyond the apex.
- Do not reuse dispensed material.

6. Warnings

- Avoid contact with eyes, skin, and oral soft tissues.
- May cause irritation or hypersensitivity reactions.

First Aid Measures:

- Eyes: Rinse immediately with plenty of water and seek medical advice.
- Skin: Wash thoroughly with soap and water.
- Oral tissues: Rinse thoroughly if accidental contact occurs.

Do not ingest. If swallowed, seek medical attention if symptoms occur.

7. Precautions

- Use with caution in patients with known sensitivities.
- Avoid extrusion beyond the apex.
- Ensure proper isolation during use.
- For professional dental use only.

8. Storage Conditions

- Store between 10°C and 24°C (50°F – 75°F)
- Keep container tightly closed after use
- Protect from moisture and direct sunlight
- Store in a cool, dry place

9. Shelf Life

2 years from the date of manufacture when stored as recommended.

10. Packaging

- 10 Applicator Tips

11. Disposal

Dispose of in accordance with local regulations applicable in the country of use.

Manufactured in India by:



Junction Road Industrial Area, Near Ram Mandir, Khurja, U.P.-203131 (INDIA)
E-mail : nidproducts@yahoo.com, website: www.nidproducts.com
Customer Care No.: 09412641804, 09219177654
CDSCO Approved Manufacturing License No. MFG/MD/2023/000377

