

PrestoMed

Calcium Hydroxide with Iodoform

Calcium Hydroxide Intracanal Dressing Paste

INSTRUCTIONS FOR USE

1. Product Description

PrestoMed Calcium Hydroxide Paste with Iodoform is a premixed calcium hydroxide paste containing iodoform. The inclusion of iodoform enhances radiopacity and antimicrobial efficacy. This non-setting formulation ensures prolonged activity of solubilized calcium hydroxide within the root canal system.

2. Composition

- Calcium Hydroxide
- Iodoform
- Water – Soluble Base

3. Intended Use

PrestoMed Calcium Hydroxide Paste with Iodoform is intended for use as a temporary or permanent root canal filling material in patients above 6 years of age.

4. Intended User

For use by dental professionals only.

5. Indications for Use

- Disinfection of root canals
- Prevention of bacterial growth
- Promotion of periapical healing
- Stimulation of apexification and apexogenesis
- Temporary or permanent filling of infected root canals
- Management of root resorption

6. Directions for Use

1. Prepare, clean, and dry the root canal thoroughly.
2. Attach a disposable applicator tip securely to the syringe and verify material flow.
3. Insert the needle into the canal, approximately 2mm short of the apex.
4. Deliver the paste by gently depressing the plunger while slowly withdrawing the needle to ensure uniform filling.
5. Remove excess material using a sterile cotton pellet.

Manufactured in India by:



Junction Road Industrial Area, Near Ram Mandir, Khurja, U.P.-203131 (INDIA)
E-mail : nidproducts@yahoo.com, website: www.nidproducts.com
Customer Care No.: 09412641804, 09219177654
CDSCO Approved Manufacturing License No. MFG/MD/2023/000377

Important:

- Avoid prolonged or repeated contact with skin and eyes.
- Avoid inhalation of vapors.
- Do not ingest.

7. Warnings

- Avoid prolonged or repeated contact with skin and eyes.
- Avoid inhalation of vapors.
- Do not ingest.

Potential Effects:

May cause irritation, burns, or hypersensitivity reactions.

First Aid Measures:

- **Skin contact:** Wipe immediately with cotton and alcohol, then wash thoroughly with soap and water.
- **Eye contact:** Rinse immediately with plenty of clean water and seek medical attention.

Discontinue use if irritation, sensitization, or allergic reaction occurs and consult a physician.

Do not use in patients with known hypersensitivity to any component.

8. Precautions

- If irritation occurs to oral mucosa, rinse immediately with copious amounts of water.
- Use rubber dam isolation during application.
- Wear appropriate personal protective equipment (gloves, protective eyewear, and clothing).
- Avoid extrusion of material beyond the apex.

9. Contraindications

Do not use in patients with known allergy or hypersensitivity to calcium hydroxide, iodoform, or any other component of the product.

10. Storage Conditions

- Store between 10°C (50°F) and 24°C (75°F)
- Keep container tightly closed after use
- Protect from moisture and direct sunlight
- Store in a cool, dry place

11. Shelf Life

2 years from the date of manufacture when stored as recommended.

12. Packaging

- 2 × 3g Syringe
- 10 Applicator Tips

13. Disposal

Dispose of the product in accordance with local regulations applicable in the country of use.

