

PrestoMed

Product Description

Prestomed EDTA Solution is a 17% EDTA formulation adjusted to a neutral pH. It is designed for use during root canal procedures to aid in effective cleansing and enlargement of canals.

The solution assists in removing inorganic material from canal walls, facilitates instrumentation, and supports smear layer removal. It is particularly useful in managing calcified or narrow canals.

Composition

- Disodium EDTA
- Purified Water
- Excipients

Intended Use

To be used as an irrigating solution during endodontic (root canal) treatment.

Indications for Use

- Irrigation during root canal preparation
- Removal of smear layer
- Assistance in opening calcified canals

Directions for Use

- Transfer the required amount of solution into a disposable syringe fitted with a suitable needle.
- Gently introduce the needle into the canal until slight resistance is felt, then withdraw slightly (approximately 1–2 mm).
- Deliver the solution slowly into the canal to flush out debris and condition the canal walls.
- Do not return unused solution to the original container.

Manufactured in India by:



Junction Road Industrial Area, Near Ram Mandir, Khurja, U.P.-203131 (INDIA)
E-mail : nidproducts@yahoo.com, website: www.nidproducts.com
Customer Care No.: 09412641804, 09219177654
CDSCO Approved Manufacturing License No. MFG/MD/2023/000377

EDTA Solution

Root canal Preparation Solution

- Discard any remaining solution after use.

Warnings

- Avoid contact with skin, eyes, and soft tissues.
- Do not swallow or inhale.
- May cause irritation or hypersensitivity reactions on contact.
- In case of skin exposure, clean immediately and wash thoroughly with soap and water.
- If irritation or allergic reaction occurs, discontinue use and seek medical advice.
- In case of eye contact, rinse thoroughly with water and consult a physician.

Precautions

- Use with care to avoid extrusion beyond the apex of the canal.

Contraindications

- Not recommended for patients with known hypersensitivity to any of the ingredients.

Storage Conditions

- Store between 10°C and 24°C
- Keep the container tightly closed after use
- Protect from moisture
- Product remains stable after opening when stored under recommended conditions

Shelf Life

2 years from the date of manufacture

Packaging

- 20 ml glass bottle with dispensing dropper

Disposal

Dispose of the product in accordance with applicable local regulations.

