

PrestoMed

Gutta Percha Solvent

Solvent for Root canal Sealer

Product Description

Prestomed GP Point Solvent is a natural, effective solution designed for softening and removing gutta percha and other root canal filling materials during endodontic retreatment. It is also suitable for cleaning endodontic instruments. Formulated with d-limonene derived from natural orange oil.

Composition

- d-Limonene

Intended Use

For softening and removal of gutta percha and other root canal filling materials during endodontic retreatment procedures.

Indications for Use

- Removal of gutta percha during root canal retreatment
- Dissolution of root canal filling materials including zinc oxide–eugenol cements
- Cleaning of endodontic instruments

Instructions for Use

1. Dispense a small amount of Prestomed GP Point Solvent into a dappen dish or directly onto a cotton pellet.
2. Place the soaked cotton pellet into the root canal over the filling material.
3. Seal the access cavity with a temporary restorative material.
4. Allow the solvent to act for approximately 48 hours.
5. Re-enter the canal and remove the softened filling material using appropriate endodontic instruments.

Warnings

- Avoid contact with skin, eyes, and mucosal tissues
- Do not ingest or inhale vapors
- Prolonged or repeated exposure may cause irritation or allergic reactions

In case of exposure:

- **Skin:** Clean immediately with cotton and alcohol, then wash with soap and water
- **Eyes:** Rinse thoroughly with water and seek medical attention
- Discontinue use if irritation or hypersensitivity occurs

Precautions

- Flammable – keep away from heat, sparks, and open flames
- Use in a well-ventilated area
- Keep container tightly closed after use
- Use appropriate personal protective equipment (gloves, eyewear)

Storage Conditions

- Store between 10°C – 24°C (50°F – 75°F)
- Keep tightly closed when not in use
- Protect from moisture, heat, and direct sunlight

Shelf Life

- 2 years from the date of manufacture

Packaging

- 1 × 20 ml Glass Bottle
- 1 × Dispensing Dropper

Disposal

Dispose of in accordance with local biomedical and chemical waste regulations.

Manufactured in India by:



Junction Road Industrial Area, Near Ram Mandir, Khurja, U.P.-203131 (INDIA)
E-mail : nidproducts@yahoo.com, website: www.nidproducts.com
Customer Care No.: 09412641804, 09219177654
CDSCO Approved Manufacturing License No. MFG/MD/2023/000377

