

PrestoMed[®]

CLINICAL PRODUCTS

Trusted **Solutions.**
Superior **Outcomes.**

Advanced Dental Products
for Every Clinical Need.



Clinically Trusted
Products You Can
Rely On



Advanced Formulations
For Better Treatment
Outcomes



Easy to Use
Designed for Everyday
Clinical Excellence



Consistent Quality
Every Time

PRODUCT CATALOGUE

CLINICAL SEGMENT

NEWLY LAUNCHED



PRESTOMAX LIGHT CURE COMPOSITE NANO HYBRID

Universal Restorative Composite

Trusted Performance. Aesthetic Results.
For Superior Clinical Outcomes.



Reliable & Versatile

Ideal for Anterior
& Posterior
Restorations.



Easy to Use

Smooth Handling for
Excellent Adaptation
& Polishability.

4g



Excellent Aesthetic

Natural Shade
Blending with
High Polish Finish.



High Strength

Wear Resistant
& Durable
Restorations.



NANO HYBRID

Nano Hybrid Technology
Superior Strength,
Low Shrinkage,
Long Lasting Restorations.

HOW TO USE



- 1 Prepare the cavity and apply etchant, rinse, and dry.



- 2 Apply PrestoMax Light Cure Composite in increments (max 2mm).



- 3 Light cure each layer as per recommended curing time.



- 4 Finish and polish for a smooth, natural result.

PrestoMax

The Perfect Choice
for a Perfect Smile.



Consistent Quality
You Can Trust



Endodontist
Recommended




Designed for
Better Clinical Results

PrestoMed ETCHANT

37% Phosphoric Acid Gel

Consistent Etching. Reliable Bonding.
For Superior Adhesion.

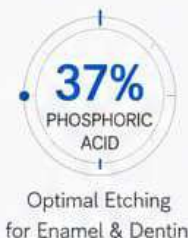

3g



Superior Etching

Creates Ideal Micro Retention for Stronger Adhesion



Thixotropic Gel

Non-Drip Formula for Precise Application



HOW TO USE



- 1 Apply PrestoMed Etchant evenly on enamel and dentin.



- 2 Leave for 15–20 seconds.



- 3 Rinse thoroughly with water for 10–15 seconds.



- 4 Gently air dry. Do not desiccate.



Consistent Etching

Ensures Uniform Etching for Better Bond Strength



Ideal Viscosity

Controlled Flow for Easy & Precise Use

PrestoMed ETCHANT
The **Perfect Start**
for a **Perfect Bond**.



Consistent Quality
You Can Trust



Endodontist
Recommended



Designed for
Better Clinical Results



Scan to view
the product
catalogue

PrestoMed EDTA GEL

Root Canal Preparation Gel

Clean. Prepare. Preserve.
For Superior Endodontic Outcomes.



Optimum Chelation
for Effective Cleaning

3g



Removes Smear Layer

Opens Dentinal Tubules
for Better Penetration



Water Soluble

Easy to Use,
Easy to Rinse



HOW TO USE



- 1 After biomechanical preparation, rinse the canal.



- 2 Apply PrestoMed EDTA Gel into the canal using a syringe or lentulo spiral.



- 3 Leave in the canal for 1-2 minutes.



- 4 Rinse thoroughly with saline or water and dry.



Optimal Canal Preparation

Facilitates Glide Path &
Enhances Instrumentation



Improved Disinfection

Enhances the Effectiveness of
Irrigants & Medicaments

PrestoMed EDTA Gel
The **Perfect Start**
for a **Perfect Finish.**



Consistent Quality
You Can Trust



Endodontist
Recommended



Designed for
Better Clinical Results




Scan to view
the product
catalogue

PrestoMed CALCIUM HYDROXIDE PASTE

Calcium Hydroxide
Intracanal Dressing Paste

Trusted Protection. Effective Healing.
For Superior Clinical Outcomes.

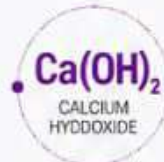

Antibacterial Action
& High pH for
Effective Disinfection.

3g



Promotes Healing

Creates an Alkaline
Environment Conducive
to Healing.



Radiopaque Formula

Easy to Verify
Placement.



HOW TO USE



- 1 Express PrestoMed Calcium Hydroxide Paste into the canal using a lentulo spiral or suitable applicator.



- 2 Fill the canal to working length.



- 3 Seal with a temporary restoration.



- 4 For best results, follow with final restoration.



Soothing & Protective

Helps Soothe Irritated
Tissues & Protect
Against Irritants.



Easy to Use

Smooth Consistency for
Effortless Placement
& Optimum Results.

PrestoMed
The **Perfect** Choice
for a **Perfect Healing**.



Consistent Quality
You Can Trust



Endodontist
Recommended



Designed for
Better Clinical Results




Scan to view
the product
catalogue

PrestoMed CALCIUM HYDROXIDE PASTE with Iodoform

Calcium Hydroxide
Intracanal Dressing Paste

Trusted Protection. Effective Healing.
For Superior Clinical Outcomes.

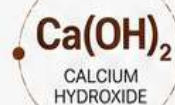

Antibacterial Action
& High pH for
Effective Disinfection.

3g



Promotes Healing

Creates an Alkaline
Environment Conducive
to Healing.



Radiopaque Formula

Easy to Verify
Placement.

HOW TO USE



- 1 Express PrestoMed Calcium Hydroxide Paste with Iodoform into the canal using a lentulo spiral or suitable applicator.



- 2 Fill the canal to working length.



- 3 Seal with a temporary restoration.



- 4 For best results, follow with final restoration.



Soothing & Protective

Helps Soothe Irritated
Tissues & Protect
Against Irritants.



Easy to Use

Smooth Consistency for
Effortless Placement
& Optimum Results.

PrestoMed
The **Perfect** Choice
for a **Perfect Healing**.



Consistent Quality
You Can Trust



Endodontist
Recommended



Designed for
Better Clinical Results



Scan to view
the product
catalogue

PRESTOMAX[®]

LIGHT CURE FLOWABLE COMPOSITE

NANO HYBRID

Precision Flow. Superior Strength.

Advanced nano hybrid technology for seamless adaptation, excellent flow and long lasting restorations.

For Superior Clinical Outcomes.



NANO HYBRID TECHNOLOGY

Superior strength, excellent flow and optimal adaptation for every cavity.



HIGH STRENGTH & DURABILITY

Wear resistant and durable restorations that last.



EXCELLENT FLOW & EASY HANDLING

Low viscosity for excellent adaptation in narrow cavities and undercuts.



HIGH ESTHETICS

Natural shade with superior polishability for a beautiful, lasting finish.



HOW TO USE



- 1 Prepare the cavity and apply etchant, rinse, and dry.



- 2 Apply PrestoMax Light Cure Flowable Composite.



- 3 Light cure as recommended.



- 4 Finish and polish for a smooth, natural result.



PRECISE ADAPTATION

Excellent flow for deep cavities and hard-to-reach areas.



EASY TO USE

Smooth handling for efficient placement and time saving.



TIME SAVING

Light cure technology ensures faster and reliable results.

PrestoMax

The Perfect Choice
for a Perfect Smile.



Consistent Quality
You Can Trust



Endodontist
Recommended



Designed for
Better Clinical Results

PrestoMed MTA

Mineral Trioxide Aggregate

INDICATED FOR CAVITY LINING/BASE, PULP-CAPPING, PULPOTOMIES, ROOT-END & PERFORATION FILLING, ROOT RESORPTION, OBTURATION & SEALING AND APEXIFICATION



Bioactive & Biocompatible
for Superior Results



Bioactive Performance

Promotes healing and formation of hydroxyapatite for long-term success.



Excellent Seal

Provides superior sealing ability against microleakage and bacterial penetration.



Optimal Working Time

Easy to mix, place and adapt with ideal setting time.



Biocompatible & Safe

Non-toxic, promotes tissue regeneration and is safe for clinical use.



HOW TO USE



- 1 Mix PrestoMed MTA powder with liquid as per requirement.



- 2 Apply the mixed MTA to the desired area.



- 3 Allow to set as per clinical requirement.



- 4 Proceed with restoration as recommended.



Versatile Indications

Ideal for a wide range of endodontic & restorative procedures.



Trusted Quality

Consistent performance you can rely on, every time.

PrestoMed MTA
Strength in **Science**.
Confidence in Every Case.



Consistent Quality
You Can Trust



Endodontist
Recommended



Designed for
Better Clinical Results



Scan to view
the product
catalogue

NEWLY LAUNCHED

PrestoMed Sodium Hypochlorite Solution

5% Sodium Hypochlorite Solution

Powerful Irrigation.
Trusted in Dentistry.



Effective Irrigation

Dissolves organic tissue and helps in cleaning the root canal system.



Reliable & Consistent

Ensures effective antimicrobial action for better outcomes.



Easy to Use

Convenient handling for everyday practice.



For use only
in Dentistry

HOW TO USE

1



Access the canal and prepare as per standard protocol.

2



Irrigate copiously with Sodium Hypochlorite solution during instrumentation.

3



Allow it to act for 1-2 minutes.

4



Rinse thoroughly with water or saline and proceed with obturation.



Helps in dissolving organic tissue, deodorizing and improving cleaning efficiency.



Effective Cleaning

Excellent tissue dissolution and canal debridement.



Antimicrobial Action

Helps reduce microbial load in the root canal system.



Better Outcomes

Enhances success rate of endodontic treatments.



Consistent Quality
You Can Trust



Endodontist
Recommended



Designed for
Better Clinical Results



Scan to view
the product
catalogue

NEWLY LAUNCHED

PrestoMed Retraction Solution

25% Aluminium Chloride Solution

Effective Retraction.
Trusted in Dentistry.



Effective Retraction

Helps in achieving clear & dry sulcus.



Reliable & Consistent

Ensures predictable retraction for better outcomes.



Easy to Use

Convenient handling for everyday practice.



For only use
in Dentistry

HOW TO USE

1



Isolate and clean the tooth to be treated.

2



Painlessly place a cotton pellet soaked with Retraction Solution in the sulcus.

3



Allow it to act for 1-2 minutes.

4



Remove the cotton pellet and rinse thoroughly with water.



Helps in gingival retraction for clear impression and better restoration.



Optimal Retraction

Provides clear access to subgingival areas



Reduces Bleeding

Astringent action helps control gingival bleeding



Better Precision

Enhances accuracy of impression & restoration



Consistent Quality
You Can Trust



Endodontist
Recommended



Designed for
Better Clinical Results



Scan to view
the product
catalogue

NEWLY LAUNCHED

PrestoMed CMCP

Camphor Mono-Chlorophenol

Effective Antiseptic.
Trusted in Dentistry.



Antiseptic Action

Helps in disinfecting the root canal system.



Effective & Reliable

Consistent performance for predictable results.



Easy to Use

Convenient handling for everyday practice.



For only use
in Dentistry

HOW TO USE

1



Prepare and clean the canal.

2



Apply PrestoMed CMCP in the root canal.

3



Allow it to act for 1–2 minutes.

4



Rinse thoroughly with saline or water and dry.



Helps in disinfection and better endodontic outcomes.



Effective Disinfection

Helps eliminate microorganisms.



Reduces Infection Risk

Supports successful endodontic treatment.



Better Clinical Outcomes

Enhances healing and treatment success.



Consistent Quality
You Can Trust



Endodontist
Recommended



Designed for
Better Clinical Results



Scan to view
the product
catalogue

NEWLY LAUNCHED

PrestoMed Citric Acid Solution

20% Citric Acid Solution

Effective Chelation.
Trusted in Dentistry.



Effective Chelation

Removes smear layer and opens dentinal tubules.



Reliable & Consistent

Ensures predictable results for improved treatment outcome.



Easy to Use

Convenient handling for everyday practice.



For only use
in Dentistry

HOW TO USE

1



Prepare and clean the canal.

2



Apply PrestoMed Citric Acid Solution on a cotton pellet.

3



Allow it to act for 10–15 minutes.

4



Rinse thoroughly with saline or water and dry.



Helps in removing smear layer and enhancing adhesion.



Smear Layer Removal

Effectively removes smear layer for better bonding.



Improved Adhesion

Enhances bonding of sealers and obturation materials.



Better Clinical Outcomes

Supports successful endodontic treatment.



Consistent Quality
You Can Trust



Endodontist
Recommended



Designed for
Better Clinical Results



Scan to view
the product
catalogue

NEWLY LAUNCHED

PrestoMed EDTA Solution

Root Canal Preparation Solution

Effective Chelation.
Trusted in Dentistry.



Effective Chelation

Chelates calcium ions and helps in removing the smear layer.



Reliable & Consistent

Ensures consistent performance for better clinical results.



Easy to Use

Convenient handling for everyday practice.



For only use
in Dentistry

HOW TO USE

1



Prepare and clean the canal.

2



Apply PrestoMed EDTA Solution on a cotton pellet.

3



Allow it to act for 1-2 minutes.

4



Rinse thoroughly with saline or water and dry.



Helps in smear layer removal and opens dentinal tubules.



Smear Layer Removal

Helps in removing smear layer for better cleaning.



Enhanced Chelation

Chelates calcium ions for better canal preparation.



Better Clinical Outcomes

Supports effective cleaning and improved outcomes.



Consistent Quality
You Can Trust



Endodontist
Recommended



Designed for
Better Clinical Results



Scan to view
the product
catalogue

NEWLY LAUNCHED

PrestoMed Pain Relief Oil

Analgesic Antiseptic Solution

Effective Pain Relief.
Trusted in Dentistry.



Pain Relief

Provides immediate relief from pain and sensitivity.



Antiseptic Protection

Helps in preventing infection in the treated area.



Easy to Use

Convenient handling for everyday practice.



For only use
in Dentistry

HOW TO USE

1



Isolate and clean the area.

2



Apply PrestoMed Pain Relief Oil on a cotton pellet.

3



Place the cotton pellet on the affected area to relieve pain.

4



Allow it to act for a few minutes.



Provides relief from pain and helps maintain aseptic conditions.



Fast Pain Relief

Quick and effective analgesic action.



Antiseptic & Safe

Helps maintain asepsis in the treated area.



Better Patient Comfort

Enhances overall patient experience and satisfaction.



Consistent Quality
You Can Trust



Endodontist
Recommended



Designed for
Better Clinical Results



Scan to view
the product
catalogue

NEWLY LAUNCHED

PrestoMed

Gutta Percha Solvent

Solvent for Root Canal Sealer

Effective Solvent.
Trusted in Dentistry.



Fast Dissolving Action

Effectively softens and dissolves gutta percha for easy removal.



Safe & Effective

Formulated for safe use without harming dentin or instruments.



Easy to Use

Convenient handling for everyday practice.



For only use
in Dentistry

HOW TO USE

1



Apply PrestoMed Gutta Percha Solvent into the canal.

2



Allow it to act for 1-2 minutes.

3



Remove softened gutta percha using files or pluggers.

4



Rinse thoroughly with saline or water and dry.



Helps in efficient removal of gutta percha for better retreatment outcomes.



Efficient Removal

Quickly softens gutta percha for easier removal.



Improved Retreatment

Enhances cleaning and shaping during retreatment.



Better Clinical Outcomes

Supports successful endodontic treatment.



Consistent Quality
You Can Trust



Endodontist
Recommended



Designed for
Better Clinical Results



Scan to view
the product
catalogue

NEWLY LAUNCHED

PrestoMed Prophylaxis Paste

Dental Abrasive Paste

Effective Cleaning.
Trusted in Dentistry.



Effective Cleaning

Removes plaque, stains and surface deposits for a smooth, clean finish.



Safe & Gentle

Low abrasive formula ensures safe use on enamel and restorations.



Easy to Use

Smooth consistency for efficient and comfortable prophylaxis.



**For only use
in Dentistry**

HOW TO USE

1



Apply Prophylaxis Paste on a prophylaxis cup or brush.

2



Polish the tooth surface using light pressure in circular motion.

3



Continue polishing for 10–20 seconds per area.

4



Rinse thoroughly with water and air dry the surface.



Provides clean, smooth and shiny teeth surface.



Plaque & Stain Removal
Effectively removes plaque and extrinsic stains.



Enamel Safe Formula
Gentle on enamel and restorative materials.



Better Patient Experience
Leaves the mouth feeling clean and refreshed.



Consistent Quality
You Can Trust



Endodontist
Recommended



Designed for
Better Clinical Results



Scan to view
the product
catalogue

NEWLY LAUNCHED

PrestoMed FormaCresol

Dental Pulp Fixation Solution

Effective Pulp Fixation.
Trusted in Dentistry.



Fixates Pulp Tissue
Helps in preserving
pulp vitality.



Effective & Reliable
Consistent performance
for predictable results.



Easy to Use
Convenient handling for
everyday practice.



**For only use
in Dentistry**

HOW TO USE



After biomechanical
preparation, remove
the excess moisture.



Apply PrestoMed
FormaCresol on a
cotton pellet.



Place the cotton pellet
in the canal and
allow it to act.



Remove the cotton
pellet and proceed
with obturation.



Helps in
pulp fixation
and disinfection
of the root
canal system.



Optimal Pulp Fixation
Supports pulp tissue
preservation



Disinfection Support
Assists in reducing
microbial load



Better Clinical Outcomes
Enhances treatment
success



Consistent Quality
You Can Trust



Endodontist
Recommended



Designed for
Better Clinical Results



Scan to view
the product
catalogue

PrestoMed[®]

CLINICAL PRODUCTS

Advancing Clinical Dentistry Through Innovation

At OrthoPlast, we are committed to developing high-quality clinical dental solutions designed to support predictable outcomes, efficient workflows, and superior patient care. The PrestoMed[®] range is trusted by dental professionals for its reliability, performance, and consistency across a wide spectrum of clinical applications.



Trusted **Solutions.**
Superior **Outcomes.**

OUR COMMITMENT



Quality-Driven
Manufacturing



Scientifically
Developed
Formulations



Clinically
Proven
Performance



Consistent
Product
Reliability



Professional
Support &
Service

CLINICAL PORTFOLIO



Endodontic Solutions



Restorative Materials



Preventive Care Products



Clinical Consumables



Specialty Dental Formulations

Designed for **Better Clinical Results.**



MANUFACTURED BY - ORTHOPLAST



Junction Road, Near Ram Mandir,
Khurja - 203131 (U.P.) - INDIA



+91 9412641804 | +91 9219177654
+91 9810654271 | +91 8630079165



nidproducts@gmail.com

FOLLOW US

